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Instructions for use

Please read this information carefully because it will help you use HYLO CARE® properly.

HYLO CARE®

Sodium hyaluronate 1 mg/ml and dexpanthenol

For lubrication and support of healing of a damaged surface of the eye e.g. due to dry eyes or surgical procedures.

Sterile, phosphate-free eye drops without preservatives

What is HYLO CARE®?

HYLO CARE® is a sterile preservative-free solution for ocular use.

Composition:

1 ml HYLO CARE® contains 1 mg sodium hyaluronate, dexpanthenol, citric acid anhydrous, sodium citrate and water for injections.

What is HYLO CARE® used for?

For lubrication and support of healing of a damaged surface of the eye e.g. due to dry eyes or surgical procedures.

When should you use HYLO CARE®?

In order to restore and maintain healthy eyes the ocular surface - cornea and conjunctiva - must be kept permanently wet. Ideally this function is fulfilled by our tears. Occasionally, however, there are not enough tears or their composition is disturbed to which the eye responds with redness, itching, foreign body and burning sensation, or acute tiredness. These symptoms of an inadequate supply with physiologic tears may occur in dry or air-conditioned environments, as a result of medical treatment or – as a sign of age.

The epithelial cells of cornea and conjunctiva are continuously restored and this process works best in a humid environment. It is therefore vital, also after surgical procedures, to keep the ocular surface sufficiently lubricated to allow proper wound healing.

In all these cases it is advisable to apply HYLO CARE® which supplies intensive humidity to the eye restoring its natural moisture balance.


HYLO CARE® contains the sodium salt of hyaluronic acid which is a physiologic substance of the eye and of other parts of the body. Sodium hyaluronate has the unique property of forming an even, stable and long lasting moisturizing film on the surface of the eye without being washed away quickly. This prolongs the moisturizing effect without impairing visual acuity. Therefore HYLO CARE® can be used during the day as compared to eye ointments and gels which cause blurred vision because of their greasy consistency.

Additionally HYLO CARE® contains dexpanthenol which belongs to the extensive group of B vitamins. This component fulfils the eye's need for intensive regeneration while supplementing the lubricating properties of sodium hyaluronate. In addition, Dexpanthenol promotes wound healing and so both superficial erosions of the ocular surface resulting from an insufficient tear film as well as surgical wounds heal faster.

HYLO CARE® thus ensures sufficient moistening of the eye and at the same time enhances the regeneration of corneal and conjunctival epithelia.

How to apply HYLO CARE®?


Only use HYLO CARE® if the sealing of the folding carton was intact before the first use (☞).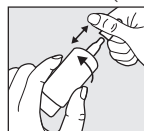


Step 1:

Remove cap before use.

For this purpose, hold the cap right above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement.

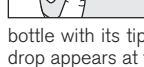
Before the first application please hold the HYLO CARE® bottle with its tip down and press down the bottom of the bottle until the first drop appears at the dropper tip. Now the bottle is ready for use.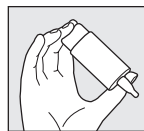


Step 2:

Hold the bottle upside down with your thumb on the shoulder of the bottle and the other fingers on the bottom of the bottle.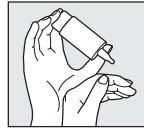


Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.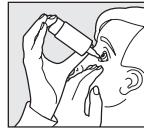

Step 4:

Slightly tilt back your head, gently pull down your lower eyelid and press down the centre of the bottle base vigorously and quickly. This activates the mechanism to release one drop. Because of the special COMOD® valve system, size and speed of each drop remain constant independent of how much pressure is applied. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

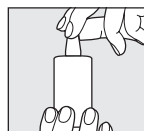Step 5:

Replace cap immediately and carefully after use. Take care that the dropper tip is dry.

When applying the fluid please avoid any contact between the tip of the bottle and your eye or your skin.

The COMOD-bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount of solution remains in the bottle beyond its use.

Only one person should be treated with one particular bottle of HYLO CARE®.

What else do you need to know?

If you use any other eye drops allow for at least 30 minutes between applications and always apply HYLO CARE® last. Eye ointments should, however, always be administered after using HYLO CARE®.

Being preservative-free, HYLO CARE® is well tolerated even in long-term applications. In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately after the use of HYLO CARE® was discontinued.

HYLO CARE® may be used while wearing contact lenses. After inserting contact lenses we recommend to allow for about 30 minutes before applying the eye drops to avoid intolerance reactions which may occur due to interactions between lens care products and HYLO CARE®.

On the basis of the available findings HYLO CARE® may be used in children, pregnant and breast-feeding women.

How often and for how long can you use HYLO CARE®?

HYLO CARE® should be used according to your condition and the instructions given by your doctor or your pharmacist. Generally you instil one drop of HYLO CARE® three times a day into each eye. If necessary you may use it more frequently. If you use HYLO CARE® more than 10 times per day please consult your eye doctor. HYLO CARE® is suitable for long-term treatment. Nevertheless, as with all complaints you should consult your eye doctor if symptoms persist.

When should you not use HYLO CARE®?

Do not use HYLO CARE® if you are hypersensitive to one of the ingredients.

What you need to know about storing HYLO CARE®?

Do not store above 25 °C (☞^{25 °C}).

HYLO CARE® may be used for 6 months after opening.

Do not use beyond the expiry date ☞.

Keep out of the reach of children.

The following packaging sizes are available:

10 ml eye drops
2 x 10 ml eye drops (only in Germany)

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Medical device

Your opinion about our product is of great importance for us!
We would be pleased to receive your feedback via post, fax (no. +49 6805/9292-222)
or via www.ursapharm.info/hylocare.

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URSAPHARM Arzneimittel GmbH,